



DIRECTIONS FOR USE

VELOCITY® ORTHOPEDICS CLEAR 'N DRY™ ARTHROSCOPIC CANNULA

Device Description

The Velocity Orthopedics Clear 'N Dry™ Arthroscopic Cannula system consists of a transparent cannula with a latex-free fluid seal and a conical obturator. The cannula threads are designed to hold the cannula in place. The Clear 'N Dry™ Arthroscopic Cannula is supplied sterile and is for single use only.

Indications for Use

The Clear 'N Dry™ Arthroscopic Cannula is intended to maintain portals for the utilization of instrumentation during arthroscopic surgery.

Contraindications

Known hypersensitivity to the material.

Warnings

Do not use if the product sterilization barrier or product packaging is compromised or damaged.

Contents are sterile unless package is opened or damaged. **DO NOT RESTERILIZE.**
For single use only.

This device should only be used by physicians with training in, and a thorough understanding of, orthopedic surgery and associated surgical techniques.

Precautions

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

It is recommended to initially use a trocar, independent of the cannula. Use care if inserting the trocar into the cannula to avoid puncturing the seal.

The cannula should be fully inserted and visible within the joint cavity to avoid the possibility of fluid extravasation into the tissue.

Verify that instruments are compatible with cannula size prior to use.

When using an arthroscope, it may be necessary to remove the arthroscope sheath to fit within the cannula.

Take care to avoid anatomical hazards.

After use, discard device according to facility's standard biohazard procedures.

Instructions for Use

The cannula fluid port has a standard Luer and 1/4" tube connection, which may be used for inflow or outflow. Irrigation may proceed with or without an instrument in place.

1. Inspect packaging, sterile barrier, ablation device, cable connections, and suction tubing (as applicable) prior to use. Do not use if product is damaged or sterile packaging is compromised.
2. Make a small incision with a scalpel.
3. Insert the blunt obturator into the cannula.
4. Insert the assembly into the incision to the desired depth by turning the assembly clockwise, then remove the obturator.
5. Visually confirm that the cannula has fully entered the joint.
6. Remove the cannula from the incision by turning the cannula counterclockwise.

HOW SUPPLIED:

The Velocity Orthopedics Clear 'N Dry™ is supplied sterile and is for single use only. Do not resterilize. The system contains no latex.

STORAGE AND HANDLING:

Store in a cool, dry place (at or below 25°C or 77°F) and in a manner that protects the integrity of the package and sterile barrier.

R_x ONLY



Read Instructions Before Use



Single Use Only



Do Not Resterilize

10° C
10° C
40° C



Sterile, ETO



Do Not Use Beyond Expiration Date



Do Not Use if Packaging Is Opened Or Damaged



STORAGE
10-75% non-condensing
OPERATE
30-70% non-condensing
TRANSPORT
10-100% including condensation



30° C STORAGE
40° C OPERATE
+70° C TRANSPORT

Not Made with Natural Latex Rubber

WARRANTY

Velocity Orthopedics, Inc. warrants against defects in both materials and workmanship to the owner at the time of purchase. All components are covered by the warranty for a period of one year from date of purchase.

Disclaimer of Warranty and Limitation of Remedy

There is no express or implied warranty, including without limitation and implied warranty of merchantability or fitness for a particular purpose, on the Velocity Orthopedics product(s) described in this publication. Under no circumstances shall Velocity Orthopedics be liable for any direct, incidental, or consequential damages other than as expressly provided by specific law. No person has the authority to bind Velocity Orthopedics to any representation or warranty except as specifically set forth herein.

Description or specification in Velocity Orthopedics' printed matter, including this publication, are meant solely to generally describe the product at the time of manufacture and do not constitute any express warranties.

Velocity Orthopedics will not be responsible for any direct, incidental, or consequential damages resulting from reuse of the product(s), where not directed to do so.



Velocity Orthopedics, Inc.
10700 Jersey Blvd. Suite 360
Rancho Cucamonga, CA 91730 USA
(909) 987-4343
www.velocityorthopedics.com